VOCO

Admira Bond

GB

CE⁰⁴⁸²

Admira in accordance with EN 24049/ISO 4049 Instructions for use

Admira Bond is an Ormocer®-based light-curing one-component dentine-enamel bond. **Admira Bond** contains specially developed adhesive Ormocer®s as well as additi-ve aliphatic and aromatic dimethacrylates. Ormocer®s as a material type are protected by patent. **Admira Bond** cures under halogen light/LED-light (blue light).

Indications: **Admira Bond** is suited for direct restoration of all filling classes in combination with light-curing restorative material.

Directions: **Cavity preparation:** In general, a minimal preparation according to the rules of the adhesive filling technique should be preferred in order to conserve as much healthy tooth substance as possible.

Cleaning/Drying:

Cleaning and drying of the cavity is carried out with special preparations or hydrogen peroxide solution. Due to the subsequent application of light-curing restorative material, drying the work field is required. Avoid contamination of the cavity with blood or saliva.

Lining: A calcium hydroxide lining should be applied close in proximity to the pulp. A lining is recommended in very deep cavities. Due to their fluoride release, glass poly-alkenoate (glass ionomer) cements are most suitable for this purpose.

Bevelling/Etching:

Use **Admira Bond** in the "total-etch-technique". Bevel the enamel and then etch the entire cavity, starting from the enamel margin (enamel approx. 20 - 30 s, dentine maximum 15 s with **Vococid**). In case of fluoridated enamel or enamel with few prisms, prolong the etching time **on the enamel only** accordingly (up to 90 s). Then remove etchant, rinse for 20 s and remove excess moisture with the air blower. The dentine surface should appear slightly moist (not too dry) and the enamel surface should have a chalky white appearance.

Application of Admira Bond: Dispense **Admira Bond** on a **Single Tim** or a disposable brush. As an alternative, dispense **Admira Bond** on a mixing pallet.

Apply **Admira Bond** evenly on all prepared enamel and dentine surfaces with **Single Tim** or a disposable brush and let act for 30 s.

Then disperse **Admira Bond** with a faint airjet and poly-merise with halogen light/LED-light for 20 s. To light-cure this material conventional polymerization devices are suited. The light output should be a minimum of 500 mW/cm² on halogen polymeri-sation devices and 300 mW/cm² on LED devices. Do not remove or contaminate the inhibition layer since it is important for the adhesion to the filling material.

Filling: Process the filling material according to the manufac-turer’s instructions for use.

Information/Precautionary measures:

- No known side effects. Hypersensitive persons may have a reaction.
- **Admira Bond** contains Ormocer®s, Bis-GMA, HEMA, BHT, acetone and organic acids. Do not use in cases of known allergies to these ingredients. Avoid contact with the oral mucosa. In case of contact of **Admira Bond** with the eyes, rinse immediately with plenty of water and consult a physician.
- Materials containing thymol, eugenol or oil of cloves interfere with the curing of filling composites and thus should not be used in combination with filling composites.

Storage: Tightly close **Admira Bond** immediately after use. Store at temperatures between 4°C - 23°C (storage at 5°C - 8°C prolongs the shelflife of the product). Do not expose **Admira Bond** to direct sunlight or the chairlight. Do not use after the expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and correspond- ing to the conditions as stipulated in our general terms of sale and delivery.

Ormocer® is a trademark of Fraunhofer Institut, Würzburg, Germany.

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VC 00 020475 E10 0210 99 © by VOCO