
VOCO Amalgam Liner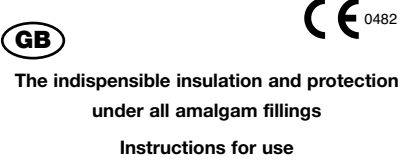

The indispensable insulation and protection

under all amalgam fillings

Instructions for use

Amalgam Liner is a varnish for the insulation of dentine against amalgam and the sealing of amalgam fillings. The silver contained in **Amalgam Liner** binds excess mercury and thus reduces its ingestion in the critical setting phase of the amalgam.

Amalgam Liner is applied:

- as a dentine protection/lining under amalgam fillings
- for the sealing of amalgam fillings immediately after insertion.

Methods of use:

Prepare cavity for filling with amalgam, clean and thoroughly dry it.

Use a lining or pulp protection in deep cavities and in the proximity of the pulp.


To shake the bottle of **Amalgam Liner**, take it between the thumb and the index; the silver must be well dispersed in the solution. Once the steel balls are moving freely, do not shake too vigorously any more.

Dispense **Amalgam Liner** drop by drop on a disposable brush or foam pellet and touch all dentine surfaces and linings of the dried cavity up to the rim. Carefully dry the varnish with the air-jet (ca. 30 s) and then place the amalgam filling. To reduce the impact of saliva and the mercury release during the setting phase the surface of a fresh amalgam filling can then additionally be sealed with **Amalgam Liner**.

Additional indications/precautions:

- Close the bottle with **Amalgam Liner** tightly immediately after use and store in the dark at room temperature (4°C - 23°C).
- Fresh stains of **Amalgam Liner** can be removed with Solvent (first test the resistance of the material), or with agents to remove developers/fixers (x-ray chemicals).
- Amalgam Liner** should always have an optimum consistency. Thickened material can be thinned with some drops of Solvent until the correct consistency is reached (compare with a new bottle).
- Cotton wool leaves threads in the cavity and is not suited to apply **Amalgam Liner**.
- Amalgam Liner** contains ethyl acetate, silver, sodium fluoride, calcium fluoride, celulose ester, eugenol; reactions on ingredients cannot be excluded.
- Solvent is inflammable. Do not use **Amalgam Liner** and Solvent in the proximity of flames!
- Amalgam Liner** is not suited for restorative materials other than amalgam, nor in the anterior area.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

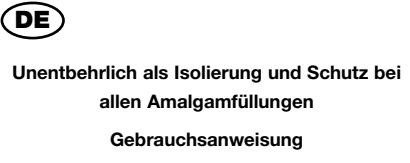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VOCO GmbH **Phone +49 (4721) 719-0**