VOCO**GB****Argion****CE**⁰⁴⁸²**Instructions for use****in accordance with ISO 9917**

Argion - the silver-reinforced glass polyalkenoate (ionomer) filling cement that chemical bonds to enamel and dentine.

Indications:

- for restoration of deciduous teeth (especially class I)
- cavity fillings with carious lesions
- filling and core build-ups, linings

Method of use:

Argion's adhesion to enamel and dentine prevents marginal leaks and creates retentions. Undercuts give additional hold to the filling. The layer thickness of the fillings should be at least 1 mm.

Clean the cavity (for instance with a 3% hydrogen peroxide) and then rinse carefully with water and dry with an air jet. If the remaining dentine is thicker than 1.5 mm, no additional lining is necessary. To avoid pulp irritations which may occur in case of insufficient dentine thickness covering of the areas near the pulp with calcium hydroxid is recommended.

Attention! Mix **Argion** only with water.

Shake powder well before use!

Apply **Argion** at room temperature (19°C - 23°C). In case of high temperatures using a cooled glass plate is recommended. Place **Argion** powder on a glass plate or suitable mixing pad and mix the powder portion by portion with water. Use 1 drop of water for 1 level measuring scoop of powder (this corresponds to a mixing ratio powder/water = 7 : 1).

When dispensing the water, hold the dropper bottle in an absolutely vertical position, making sure that the metal annula protrudes approx. 2 - 3 mm above the top. Do not use drops for mixing that flow out when bending the bottle.

A strong plastic spatula or another abrasion resistant spatula should be used for mixing. Mix on a glass plate or other nonabsorbent surface. Start mixing in large portions and continue with smaller ones.

| | | | |
|-------------|--------------|--------------|-----------|
| Mixing time | Working time | Setting time | |
| 0 | 50 - 60 s | 2 - 3 min | 4 - 5 min |

Mixing time is approx. 50 - 60 s. Do not exceed 1 min.

Working time from end of mixing procedure is approximately 2 min (room temperature) after the filling has been inserted into the cavity. **Argion**'s setting time is approximately 2 - 3 min. The working area must be kept dry during the filling procedure. Application of rubberdam is recommended. Linings and build-up fillings can be safely worked after setting (approximately 4 - 6 min after application).

Fillings should be at least 1 mm deep. Prevent feather-shaped margins. When in doubt, prepare in a conventional form.

Argion is susceptible to water contamination during the setting phase.

Immediately cover filling with a strip, matrix or a protective varnish after application. The rough excess should be removed immediately. Upon removal of the strips or matrix, the filling must be completely covered again with protective varnish. In fillings, finishing and contouring (flexible disc) should be done not earlier than 15 min after application. Repeat varnish and let cure. The best results for final polishing are obtained 24 h after the filling has been inserted.

Precautionary measures/warnings:

Close powder container immediately after use and avoid humidity to maintain the properties of the material. If the thickness of the remaining dentine layer is insufficient, pulp irritations cannot be excluded.

Argion contains polyacrylic acid, fluoride silicate and silver. Do not use in cases of known sensitivities or allergies.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve the user from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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