**VOCO**

## Calcimol

**GB**

#### Instructions for use

**Calcimol** is a self-curing radiopaque calcium hydroxide paste.

Calcimol is a self-curing radiopaque calcium hydroxide paste.

**Indications:** **Calcimol** is intended for indirect pulp capping and as a liner under dental filling materials.

Calcimol is a self-curing radiopaque calcium hydroxide paste.

**Method of use:**

Mix equal quantities of **Calcimol** base and catalyst (mixing ratio 1.2 : 1 g/g) on a mixing pad or glass plate for approx. 10 s. The mixture should achieve a homogeneous consistency and uniform colour. Do not mix for too long! Once mixed, insert **Calcimol** into the cavity as quickly as possible.

Use **Calcimol** at room temperature (15°C - 23°C). At room temperature **Calcimol** remains workable for approx. 2 min and cures in the mouth within 2 - 4 min. Higher temperatures and humidity accelerate the setting time and reduce the working time, causing **Calcimol** to set more quickly in the patient's mouth.

**Supplementary/precaution instructions:** **Calcimol** contains calciumhydroxide, salicylate, calcium wolframate, N-ethyltoluol-sulfonamide: do not apply in cases of sensitivities or allergies against any component. Let totally cure before filling in order to reach a sufficient stability for filling. Do not use **Calcimol** as temporary filling or as the only liner carrying masticatory forces. The catalyst hardens at low temperatures. Allow to reach room temperature before application, if necessary by shortly heating to 40°C. **Calcimol** has a strong alkaline effect, absolutely avoid contact with eyes (mucosa). Avoid contact with etchants (solubility).

**Storage:** Store **Calcimol** at room temperature (15°C - 23°C). Reclose the tubes carefully after use. Be certain to match the colour-coded caps to the proper tube. Avoid humidity.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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VC 60 AA1097 E1 0310 99 © by VOCO
**VOCO GmbH**