Ionofil Plus AC is a radiopaque glass polyalkenoate (ionomer) cement in Application Capsules (AC) for direct application of fillings.

Ionofil Plus AC chemically adheres to enamel and dentine and continuously releases fluoride ions.

Ionofil Plus AC is available in shades A1, A3 and B3. Each Application Capsule contains 0.19 ml of cement.

Indications:

- restoration of deciduous teeth (especially class I)
- repair of wedge-shaped defects, enamel and root erosions in the dental neck area including class V fillings
- cavities or carious lesions
- class III fillings and small class I fillings (incl. extended fissure sealing) as well as core build-up under crowns and as a cavity liner.


Application:

Preparation:

The adhesion of **Ionofil Plus AC** to enamel and dentine prevents the formation of marginal gaps and creates retentions. Undercuts give additional hold to the filling. The layer thickness of fillings should not be less than 1 mm. Avoid feather-shaped margins. When in doubt, a conventional preparation should be preferred.

Clean the cavity (e. g. 3% hydrogen peroxide), thoroughly rinse with water and dry (warm air). Do not overdry dentine. If the remaining dentine is thicker than 1.5 mm, no additional lining is necessary. To avoid pulp irritations which may occur in case of insufficient dentine thickness, cover the areas in proximity of the pulp with calcium hydroxide.

The capsule has to be activated, mixed and opened before application.

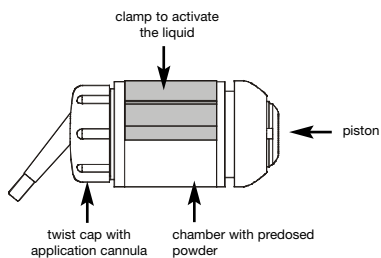Activation:

For activation, the liquid from the sachet underneath the clamp has to be pressed into the inner chamber of the capsule completely. This is done with the help of a capsule activator. Place the capsule in the activator so that the application cannula sticks out from it and the clamp faces the movable part of the activator. **Hold the lever for 2 s in the compressed position.**

Mixing:

Insert the activated capsule in a capsule mixer (4.000 - 4.500 oscillations per minute) and mix for approximately 10 s. Immediately remove the capsule from the mixer. The use of capsule mixers of other manufacturers may cause shorter or longer mixing times due to different mixing frequencies.

Opening:

Turn the twist cap with application cannula by a half turn to the right (q.v. inbassing on the cap) until you notice a stop.

Application:

Insert the opened capsule into an applicator. By turning the capsule in the applicator the cannula can be directed into the desired direction. Apply the material directly into the cavity by pressing the lever of the applicator within 30 s of mixing.

Mixing time	working time	setting time
10 s	2 min	4 min

After mixing, **Ionofil Plus AC** has a working time of approximately 2 min (room temperature). During application of a filling, the work field must be kept dry. The use of a rubber dam is recommended.

Glass ionomer cements are susceptible to humidity (breath, saliva) and drying out during the setting time, especially directly after application. If possible, cover immediately with a strip or matrix or apply a protective varnish (see instructions for use **Final Varnish LC**).

Finishing and polishing (flexible polishing discs) of fillings should be conducted no earlier than after 4 min time in the mouth. Recommendation: Re-apply protective varnish afterwards.

The best results of fine finishing will be achieved approximately 24 h after the filling has been inserted.

Additional indications/precautions:

Ionofil Plus AC should be mixed only immediately before application in order to have the full working time available for inserting and shaping, and to avoid cracks from working the material too late.

Please note that the AC capsule has to be mixed right after its activation.

Ionofil Plus AC contains polyacrylic acid, fluorosilicate and parabenes; do not use in cases of known sensitivities or allergies.

Storage:

Store **Ionofil Plus AC** mixing capsules in a dry area at temperatures between 15°C - 23°C. Do not remove foil until ready to use.

Do not use after the expiry date printed on this package.

Tightly close **Final Varnish LC** container immediately after use. Store at room temperature and protected from light.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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VOCO GmbH Phone +49 (4721) 719-0