VOCO**(GB)**

Ionolux

Application capsules

Instructions for use

Ionolux AC is a resin-reinforced light-curing radiopaque glass-polyalkenoate restorative in application capsules (AC). The material has outstanding aesthetics, good handling properties and high chemical adhesion. Each application capsule delivers a minimum of 0.14 ml of material.

Shades: A1, A2, A3**Indications:**

- Restorations class III and V, especially cervical fillings and root caries
- Fillings on deciduous teeth
- Small class I fillings
- Temporary fillings
- Core build-up
- Lining

Method of use:**Preparation:**

In principle, cavity preparation should be carried out only minimally invasive, to save healthy tooth substance. It is not necessary to prepare undercuts. Class V cavities should have a minimum depth of 1 mm. Remove preparation residues with water spray, then dry with air so that the surface remains slightly glossy (do not overdry).

Lining:

Areas close to the pulp should be protected with a calcium hydroxide lining.

Activation:

To activate the capsule, the content of the sachet under the clamp has to be completely injected into the capsule chamber. This is carried out with a capsule activator (cf. respective Instructions for use). Insert the AC capsule into the activator with the nozzle pointing upwards and the clamp pointing towards the moveable part of the activator. **Press the lever of the activator completely down and hold in this position for 2 s.** Use the material at room temperature (15°C – 23°C).

Mixing:

Fix the activated capsule in a high-frequency mixer with 4,000 – 4,500 oscillations per minute, and mix for 10 seconds. Remove capsule from the mixer immediately after the end of mixing.

Working time and consistency are influenced by the individual mixer type and may be adjusted by varying the mixing time.

Opening:

To open the capsule, turn the twist cap and nozzle clockwise through 180° until you notice a stop (see arrow on the cap).

Application:

Insert the opened capsule into an applicator (cf. respective Instructions for use). By turning the capsule in the activator, the nozzle can be directed into the desired direction. Apply the material directly into the cavity by pressing the lever of the applicator. Place a matrix, if necessary. Avoid unnecessary waiting times between mixing an application since the setting of the material may make application more complicated.

Shape the filling and then light-cure with a suitable polymerisation lamp. Fillings of more than 2 mm thickness should be applied and light-cured in layers.

At room temperature, **Ionolux AC** has a working time of 3 min. Higher temperatures shorten the working time, lower temperatures prolong it.

Light-curing:

The material can be light-cured with market-available polymerisation lamps. Their output should not be lower than 500 mW/cm². The curing time is 20 s per layer. Hold the tip of the light-curing lamp as close as possible to the surface of the filling. If the distance is more than 5 mm, the curing depth will be smaller.

Powder/liquid ratio [g/g]	2,7:1
Available amount [ml]	0.14 minimum
Mixing time [s]	10
Working time [min]	>3:00
Light-curing [s]	20

Finishing:

After light-curing, remove excess material and finish and polish the filling as usual.

Information/precautionary measures:

- Please note that the AC capsule has to be mixed right after its activation.
- In case of contact with oral tissue or skin, remove immediately with a cotton soaked in alcohol. Flush with water.
- In case of contact with the eyes, rinse immediately with plenty of water and seek medical advice.
- Ionolux AC** contains polyacrylic acid, fluorosilicate glass, amines, BHT and methacrylates. Do not use **Ionolux AC** in case of known allergies against any of the ingredients.
- Phenolic substances, especially eugenol- and thymol-containing preparations disturb the polymerisation of the restorative. Do not use zinc-oxide eugenol cements or other eugenol-containing materials together with **Ionolux AC**.

Storage:

Store at 4°C – 23°C in the unopened blister bags.

Do not use **Ionolux AC** after the expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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