VOCO**(GB)**

Polofil Supra

Instructions for use**in accordance with EN 24049/ISO 4049****(CE)**⁰⁴⁸²

Polofil Supra is a light-curing, radiopaque dental composite filling material. Due to the microfine Supra filling system, **Polofil Supra** may be used universally for restorations of anteriors and posteriors as well as for inlays. It contains 60% by volume (= 76,5% by weight) inorganic fillers, microfillers (approx. 0.05 µm) and micro particle fillers (approx. 0.5 - 2 µm). **Polofil Supra** is distinguished by its high level of stability and colour stability. **Polofil Supra** is polymerised by light (blue light).

Polofil Supra is available in practical syringes and 7 shades: reddish/brownish shades A1, A2, A3, A3.5 yellowish shade B2, B3 incisal I

Indications:

- fillings of class I, II, V in the posterior area which are not exposed to occlusal forces
- reconstruction of traumatically affected anteriors
- facing of discoloured anteriors
- colour and shape corrections for improvement of aesthetics
- locking of loosened anteriors
- extended fissure sealing for molars and premolars
- fillings of class III, IV, V in the anterior area
- repair of veneers
- core build-ups for crowns
- composite inlays

Directions:

The shade should be selected after cleaning the tooth with a fluoride-free cleaning paste. The **Polofil Supra** shade guide is made of original material and corresponds to the shade of the filling after placement and sufficient polymerisation. For better comparison, the shade guide should be moistened to improve shade matching with the tooth. Using a rubberdam is recommended.

The cavity has to be prepared according to the rules for the adhesive filling technique. To achieve an optimum marginal seal and when building up corners, the adhesive technique must be used.

In case of very deep cavities, a calcium hydroxide material should be applied.

Apply bonding agent according to the manufacturer's instructions for use.

Select the matching shade of **Polofil Supra** and apply the material.

Additional advice:

For fillings of more than 2 mm, apply and polymerise in layers (cure each layer for approx. 40 - 60 s). To light-cure this material conventional polymerisation devices are suited. The light output should be a minimum of 500 mW/cm² on halogen polymerisation devices and 300 mW/cm² on LED devices.

The curing depth, depending on the shade, is approx. 2 - 5 mm, when polymerised for 60 s. The curing depth is approx. 2 - 3 mm for translucent shades, if light-curing is done through enamel (approx. 1 mm). Transparent strips, crowns and/or light wedges should be used. Care should be taken to ensure good marginal adaption. Hold the light source of the halogen light as close as possible to the surface of the filling. If the distance is more than 5 mm, the curing depth may compromise. Polymerisation may be limited to the area of the cone of light only.

Insufficient curing can lead to discolouration and pulp irritation.

The filling can be finished and polished immediately after removing the transparent strips/crowns (fine or extra-fine grain diamond, polishing discs). During finishing and polishing the filling must be cooled. The margin of the filling, or even better the entire tooth, should be fluoridated as a final step.

Polofil Supra can be used for direct and indirect inlays according to the customary methods. The physical stability values can be improved by usual external (additional) curing.

Information:

- If the cavity cannot be dried, the prescribed technique cannot be used or in case of known allergies to certain ingredients (Bis-GMA, diurethanedimethacrylate, TEGDMA, BHT, hydroxyethylmethacrylate), **Polofil Supra** should not be used.
- Materials containing eugenol or oil of cloves interfere with the setting of filling composites, and thus should not be used in combination with filling composites.
- Cationic mouthwashes, plaque disclosing solutions and chlorhexidine may discolour **Polofil Supra**.

Storage:

Store at temperatures between 4°C - 23°C.

Syringes and bonding material bottles should be closed immediately after dispensing to prevent exposure to light. Do not use after the expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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VC 60 AA 1360 E1 0110 99 © by VOCO