VOCO**Solobond M****GB**

CE 0482

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Light-curing one-component dentine and enamel bond for a durable bond between hard substance (dentine/enamel) and light-curing restorative material without marginal gaps. **Solobond M** is used for the "wet-bonding" technique and all light-curing restorative materials that are composite based, including materials with glass ionomer content. The adhesive only has to be applied once within the total-etch technique: this is the unique feature of **Solobond M**.

Fields of application:

Solobond M is suited for direct restorations of all filling classes with light-curing restorative materials.

Application:

The adhesive bond within the total-etch technique is achieved as follows:

1. Drying

Due to the subsequent application of light-curing restorative material, it is necessary to dry the tooth surface. Avoid any contamination of the cavity with blood or saliva. The use of a rubber dam is recommended.

2. Preparation

Prepare the cavity according to the principles of the adhesive restoration technique. Areas close to the pulp should be protected with a suitable calcium hydroxide based liner.

3. Etching of enamel margins

Etch entire cavity with etching gel (**Vococid**) starting from the enamel margin (enamel: approx. 20 - 30 s, dentine: 15 s maximum). Remove etching gel by suction, rinse for approx. 20 s and remove excess moisture with an air blower. Do not overdry dentine. The surface should remain slightly moist. The etched enamel should have a chalk-white/ matt appearance. If not, repeat etching.

4. Solobond M dispensing**When using the bottle:**

Dispense **Solobond M** onto a **Micro Tim** or a disposable brush. As an alternative, dispense **Solobond M** onto a mixing pallet.

5. Solobond M application

Apply **Solobond M** homogeneously on all prepared enamel and dentine surfaces with **Micro Tim** or a disposable brush and allow to act for 30 s. Then disperse **Solobond M** with a faint air jet and polymerise with LED/halogen light for 20 s. Do not remove or contaminate the inhibition layer since it is important for the adhesion to the composite.

6. Filling

Work the filling material according to the manufacturer's instructions.

Indications, precautionary measures:

Phenolic substances (e.g. eugenol-containing temporary materials) impair polymerisation and must not come in contact with **Solobond M**. **Solobond M** contains Bis-GMA, HEMA, BHT, acetone and organic acids. In cases of known allergies against ingredients, **Solobond M** must not be applied.

Avoid contact with the oral mucosa. In case of contact of **Solobond M** with the eyes, rinse immediately with plenty of water and seek medical advice.

Micro Tim fine application brushes allow a precise and thin application.

Storage and application indications:

Tightly close **Solobond M** bottles after use. Store at temperatures between 4°C - 23°C (storage at 5 - 8°C prolongs the shelf life of the product). Do not expose **Solobond M** to direct sunlight or the chairlight.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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VOCO GmbH Phone +49 (4721) 719-0